


# CT Biopsy Grid



The Task Medical Disposable CT Biopsy Grid ensures accuracy for CT biopsy and drainage procedures across various pathologies and body parts. This simple, disposable grid saves time and money while preventing cross-contamination.

## Product Information

- Radiopaque grid for identifiable dot patterns on CT images
- 1cm markings for precise needle placement
- Flexible material conforms to patient's anatomy
- Disposable to prevent cross-contamination
- Tape strips for non-slip positioning
- Porous grid for accurate marking with a felt-tip pen
- Enhances efficiency and accuracy in CT procedures

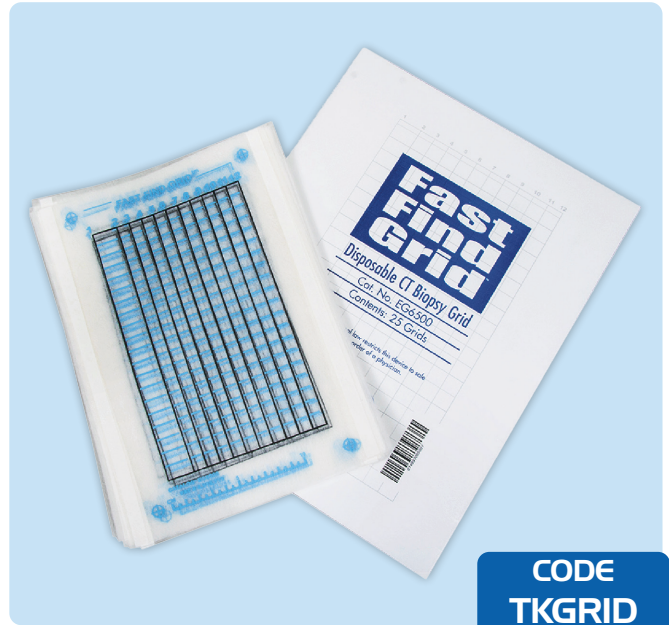

CODE  
TKGRID

## Instructions of Use

- Remove backing from tape
- Place and scan over the area of interest
- Notice the twelve dots on the skin surface
- Determine the optimal entry point
- Mark through the grid with a felt tip pen
- Remove the grid and perform the biopsy

## Packaging

- 25 grids per pouch

Base Unit



Outer Carton



[www.constar.com.au](http://www.constar.com.au)

